EXHIBIT G-1 OPERATIONAL CHECKLIST

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD OPERATIONAL PROCEDURE INTOXILYZER MODEL 8000

DUPLICATE BREATH TEST

			DATE		
			OPERATOR		
INSTRUMENT SERIAL #			LOCATION		
TEST RI	()AC)AC)AC	TIME		
Immedi	ately preceding adm	inistration of the te	ests, subject und	erwent at least a	15-minute deprivation period:
From		to		by	
	(Time)		(Time)		(Name)
() 2. () 3.	 Display reads "PUSH BUTTON TO START". Push Start Test button. Follow automated instructions on instrument display. If test record reads "Successfully Completed Test Sequence" go to step 5				
	If test record read	s "Not a Successfu		Test Sequence", a	and subject will not be tested again, go to step 5
() 5.	Remove test reco		, 1	,	
	uplicate breath tests consecutive tests sh				minutes nor more than 10 minutes apart and

DPS Form Exh G-1 (Rev 05-1)